


Setting the time and date

Setting the correct time and date in your pump is necessary for accurate basal insulin delivery and allows you to keep an accurate record of your insulin delivery and other pump functions. You can select a 12-hour or 24-hour clock. You must reset the time and date if you receive a CHECK SETTINGS alarm or you clear your pump (Clear Pump function).

1. Go to the TIME/DATE SETUP screen.
Main > Utilities > Time/Date
2. Select **12-Hour Setup** or **24-Hour Setup** and press ACT.
3. Press ACT again to change the settings.
4. Change each of the settings as follows:

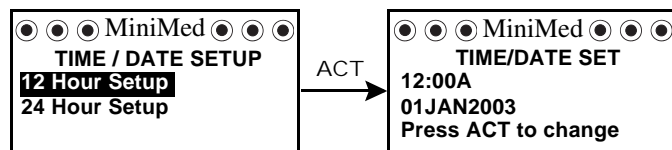

Hour	Minutes	Year	Month	Day
<div style="border: 1px solid black; padding: 5px; text-align: center;"> MiniMed SET HOUR 12:00A </div> <p>Change the hour. Press ACT.</p> <p>For 12-hour setups, press / until the correct A (am) or P (pm) appears.</p>	<div style="border: 1px solid black; padding: 5px; text-align: center;"> MiniMed SET MINUTES 9:00A </div> <p>Change the minutes. Press ACT.</p>	<div style="border: 1px solid black; padding: 5px; text-align: center;"> MiniMed SET YEAR 2003 </div> <p>Change the year. Press ACT.</p>	<div style="border: 1px solid black; padding: 5px; text-align: center;"> MiniMed SET MONTH 01 </div> <p>Change the month. Press ACT.</p>	<div style="border: 1px solid black; padding: 5px; text-align: center;"> MiniMed SET DAY 01 </div> <p>Change the day. Press ACT.</p>